

# EnteroPluri-Test



Ready-to-use system for the identification of *Enterobacteriaceae* and other oxidase-negative Gram-negative bacteria.



Liofilchem® and the Liofilchem company logo are registered trademarks of LIOFILCHEM s.r.l.

Description	Packaging	Ref.
<b>EnteroPluri-Test</b>	10 test	78618
	25 test	78619
V.P. TEST EP KOH and alpha-naphthol solutions for Voges-Proskauer test.	100/200 test	80281
KOVACS' REAGENT for indole test.	800/1000 test	80271
EnteroPluri-Test CODE BOOK	1 book	71709



Distribuciones Industriales y Científicas S.L.

**ciencia e innovación**



ref. 6552266  
Rev. 1 - 16/04/2013  
© Liofilchem 2013  
Printed in Italy

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
# Test procedure



**1**  
Remove both caps.  
Pick a well isolated colony directly with the tip of the inoculating wire.



**2**  
Inoculate EnteroPluri-Test by first twisting wire, then withdrawing wire through all twelve compartments applying a turning motion



**3**  
Reinsert wire into EnteroPluri-Test, using a turning motion through all twelve compartments, until the notch on the wire is aligned with the opening of the tube.



**4**  
The portion of the wire remaining in the tube maintains anaerobic conditions necessary for true fermentation of glucose, production of as and decarboxylation of lysine and ornithine.



**5**  
With the broken off part of the wire, punch holes through the foil covering the air inlets of the last eight compartments (adonitol, lactose, arabinose, sorbitol, Voges-Proskauer, dulcitol/PA, urea and citrate).



**6**  
Replace both caps.  
Incubate at 35 to 37° C for 18 to 24 hours with EnteroPluri-Test lying on its flat surface or in an upright position. Allow for air circulation between incubated tubes.



**7**  
Interpret and record all reactions with exception of indole and Voges-Proskauer.




**8**  
**Indole test**

**Voges-Proskauer test**

By using a syringe inject 3-4 drops of Kovacs' reagent in the H<sub>2</sub>S/Indole sector.

By using a syringe inject 3 drops of alpha-naphthol and 2 drops of potassium hydroxide in the VP sector.

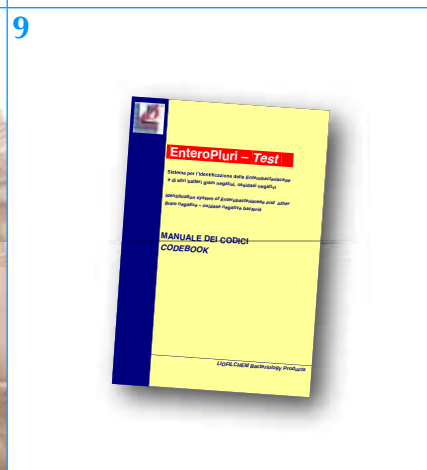

**9**  
Form the 5-digit code following the instructions and identify the microorganism using the **Codebook**.



ref. 6552266  
Rev. 1 - 16/04/2013  
© Liofilchem 2013  
Printed in Italy